



Therapeutic Listening® App
for:
听乐治疗应用程序

Apple Devices
苹果设备




Step 1 第一步

Open the “App Store”
打开“App Store”




Step 2 第二步

Search for “Therapeutic Listening”
搜索‘Therapeutic Listening’

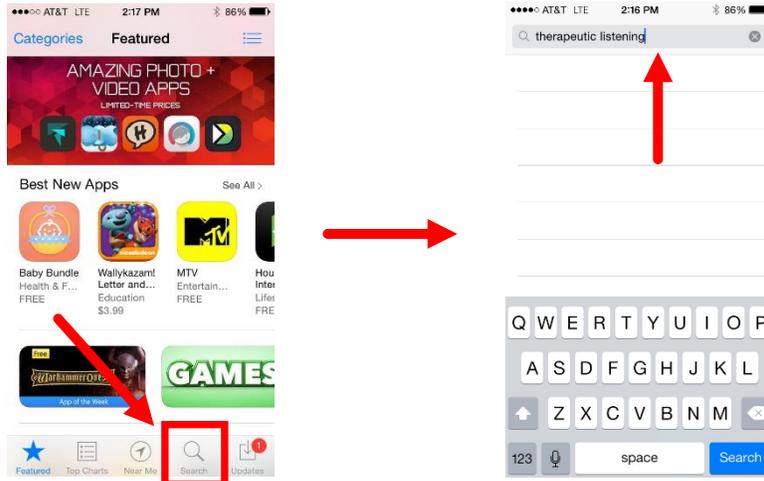

Step 3 第三步


Find the Therapeutic Listening app in the results list and select “GET” and then select
“INSTALL”
在结果列表中找到听乐治疗的应用程序，选择‘获取’，之后选择‘安装’

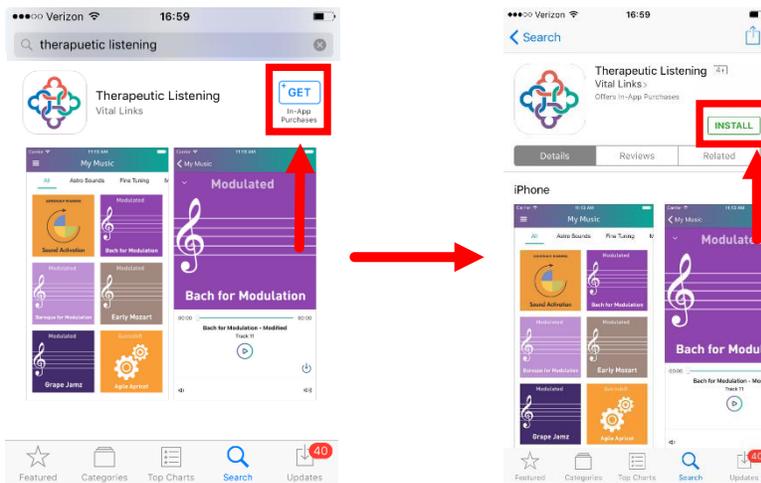


Step 4 第四步

Open the app in one of two ways:
通过下面两种方式打开应用程序:

1. Click on "Open"
while in the Store
在应用商店中按'开始'

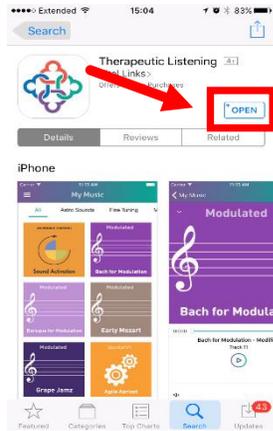


2. Locate the app on your Home Screen and
touch the icon to open
在主屏幕上找到应用程序, 然后点击图标打开

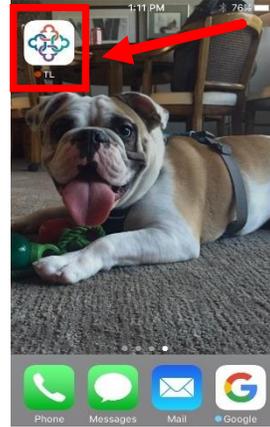

Step 5 第五步

There are two primary areas to navigate:
有两个主要的领域:


"Shop" 商店 & "My Music" 我的音乐

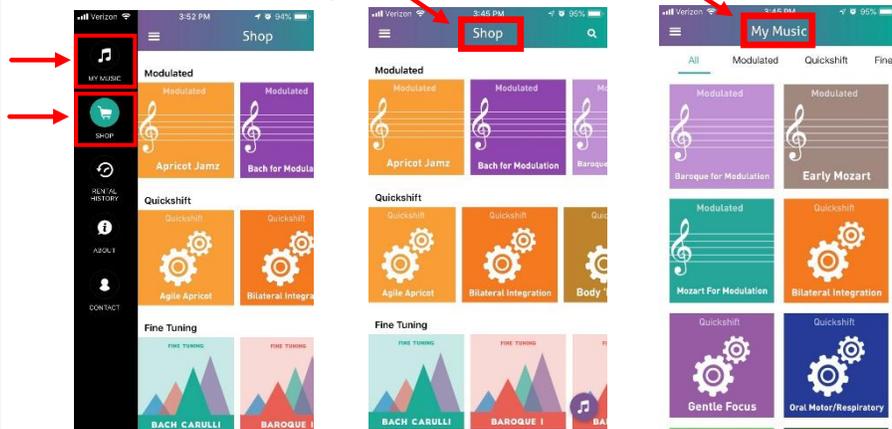

Step 6 第六步




“Shop” is where you can purchase albums. Scroll up/down to locate the Album Category you wish to search. Then, swipe the album images left/right to locate the album you wish to access.
 “商店”是你可以购买专辑的地方。向上/向下滚动找到您希望搜索的专辑类别。然后，向左/向右滑动专辑图片来定位您希望访问的专辑。

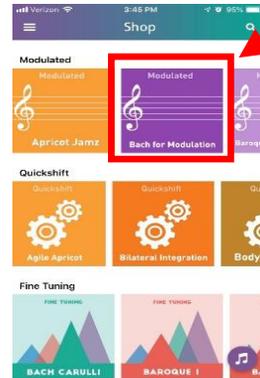


Once you locate your album, select the album image to review its details.
 一旦你找到了你的专辑，选择专辑的图片以查看其更详细的信息

Step 7 第七步

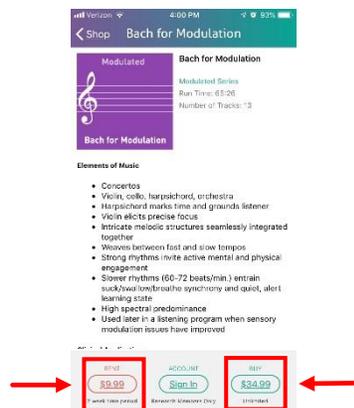


Click the “Buy” button to purchase full access to an album or you have the option to Click “Rent” to purchase access for 2 weeks*
 (*select albums only)
 点击“购买”按钮购买专辑的完整访问权限，或者点击“租借”选项购买访问权限 2 周*
 (* 仅限部分专辑)

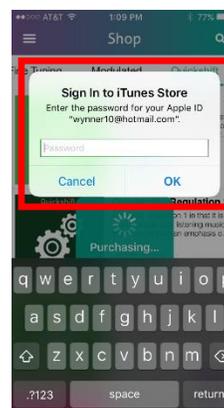

Albums are purchased/rented using your Apple iTunes ID. Transactions are considered In-App Purchases.
 使用您的苹果 iTunes ID 购买/租借专辑。交易将被视作应用内购买。


Step 8 第八步

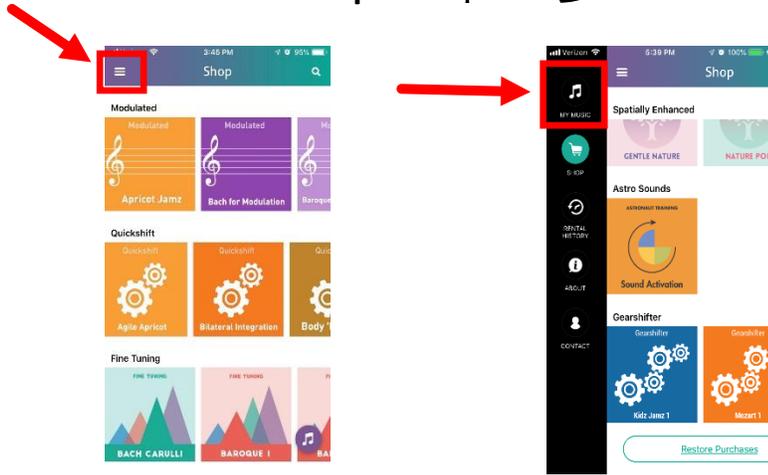

Once purchased, albums will be added to the TL App "My Music" section. To navigate to "My Music," click the menu icon (3 horizontal bars) in the top left. 一旦购买，专辑将被添加到 TL 应用程序“我的音乐”部分。要回到“我的音乐”，点击左上角的菜单图标(3个单杠)。

Enter "My Music" to access the album(s). 进入“我的音乐”以使用专辑

Step 9 第九步

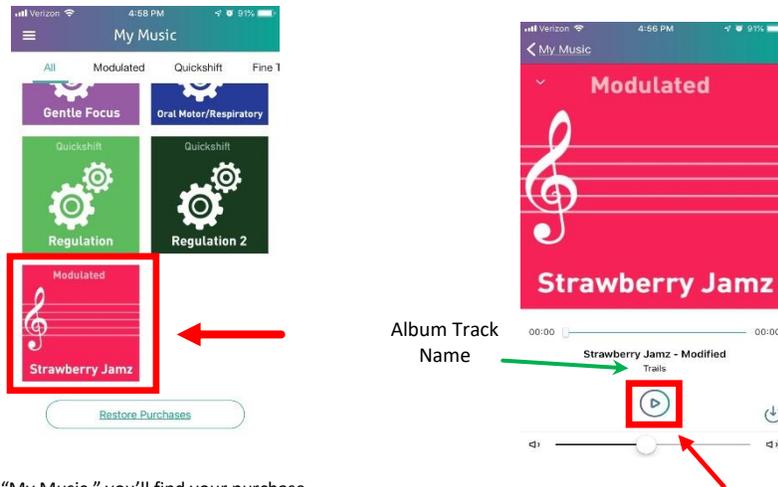

In "My Music," you'll find your purchase. Select the album image to enter the play screen. 在“我的音乐”里，你会找到你所购买的专辑。选择专辑图片进入播放界面。

Once downloaded, the "Play" button will appear and you can press it to begin the music selection. 下载完成后，“播放”按钮就会出现，你可以按下它开始聆听音乐选集。

Apple FAQs 苹果问答

- Course Completion Number Disclaimer Pop-up Message:
 - To access Modulated Music albums, a Trained Therapeutic Listening Provider Number is necessary.
 - To access Fine Tuning Music albums, an Advanced Therapeutic Listening Provider Number is required.
- 弹出需课程完成号码的免责声明:
 - 要获得调制音乐专辑，必须有一个受训过的听乐治疗提供者号码。
 - 要获得微调音乐专辑，需要一个高级听乐治疗服务提供者号码。
- A device must have the ability to download an App from the [Apple App Store](#) in order to access the TL App.
- 为了访问 TL 应用程序，设备必须有从苹果 app store 下载应用程序的能力。
- A device requires Wi-Fi or Data access to download the TL App and album purchases.
- 一个设备需要 Wi-Fi 或数据访问才能下载 TL 应用程序和购买专辑。
- A device does not require Wi-Fi or Data access to listen to purchased albums. “Airplane Mode” is recommended to avoid distraction during listening times.
- 一个设备不需要 Wi-Fi 或数据访问来收听购买的专辑。建议使用“飞行模式”，以避免在听音乐时分心。
- To access an album on multiple devices, the device must be signed into the same Apple iTunes ID account that the album was originally purchased under.
- 要在多台设备上访问一张专辑，设备必须登录到最初购买该专辑时所用的同一个苹果 iTunes ID 账户。
- The number of devices an Apple iTunes ID account can be connected to is determined by Apple; however, they typically allow up to 10 devices for an account to be signed into simultaneously.
- 一个苹果 iTunes ID 账户可以连接的设备数量由苹果决定；然而，他们通常允许一个账户同时登录最多 10 台设备。
- Album purchases are considered In-App purchases, which are not allowed as shareable items within an Apple Family Sharing Plan.
- 购买专辑被视为应用内购买，在苹果的家庭共享计划中，这是不允许共享的。
- Apple considers rentals as one-time subscriptions, which are not allowed to be shared/synced between devices, regardless of login credentials.
- 苹果将租借视为一次性订阅，不允许在设备之间共享/同步，无论登录账户是否相同。
- Albums are not transferrable between Apple and Android devices.
- 专辑不能在苹果和安卓设备之间转移。
- To regain access to your purchased albums, scroll to the bottom of “My Music” and select “Restore Purchases.”
- 重新访问购买的专辑，滚动到“我的音乐”的底部并选择“恢复购买”。
- Apple iPads, iPhones and iPod Touches are compatible with Therapeutic Listening headphones. Some devices may require a Lightning Headphone Jack Adapter to connect the headphones to the device.
- 苹果 ipad、iphone 和 iPod touch 与听乐治疗耳机兼容。有些设备可能需要一个 Lightning 耳机接口适配器来将耳机连接到设备上。
- Albums are set to shuffle automatically within the app.
- 专辑在应用程序中被自动设置为随机选择。
- Tech Tips 技术贴士:
 - Ensure your device and the TL App are updated to the most current versions.
 - Uninstall and reinstall the TL app to refresh the app, then “Restore Purchases.”
 - Further Troubleshooting:
 - Go to <https://vitallinks.com/support/contact/contact-us/> and select “Therapeutic Listening App” as your Submission Topic.
 - 确保你的设备和 TL 应用程序更新到最新版本。
 - 卸载并重新安装 TL 应用程序来刷新应用程序，然后“恢复购买”。
 - 进一步故障排除:
 - ▣ 通过 <https://vitallinks.com/support/contact/contact-us/> 并选择“Therapeutic Listening App”作为投稿主题。



Therapeutic Listening® App

for:

听乐治疗应用程序


Android Devices

安卓设备




Step 1 第一步

Open the “Play Store”
打开‘Play Store’

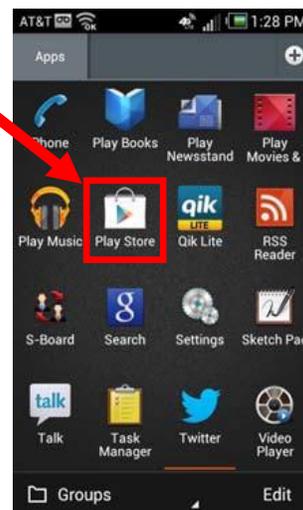


Step 2 第二步

Search for “Therapeutic Listening”
搜索“Therapeutic Listening”

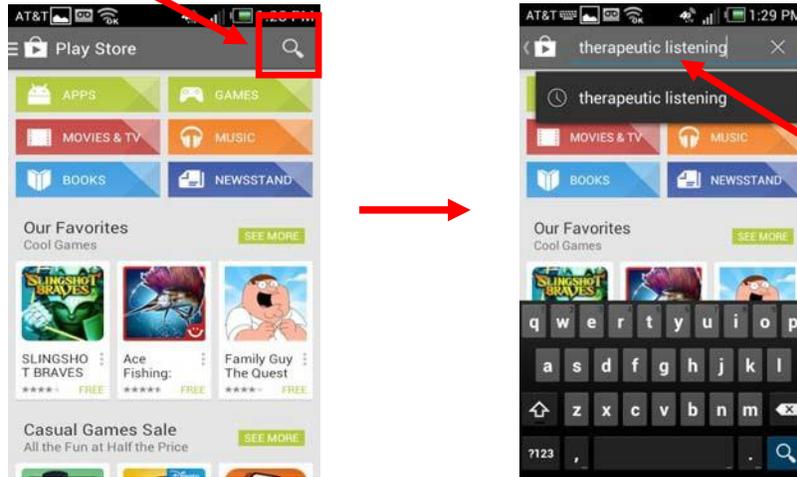

Step 3 第三步


Select the “Therapeutic Listening” app, then “INSTALL,” and then
“ACCEPT”
选择“Therapeutic Listening”应用程序，之后“安装”，之后“接受”

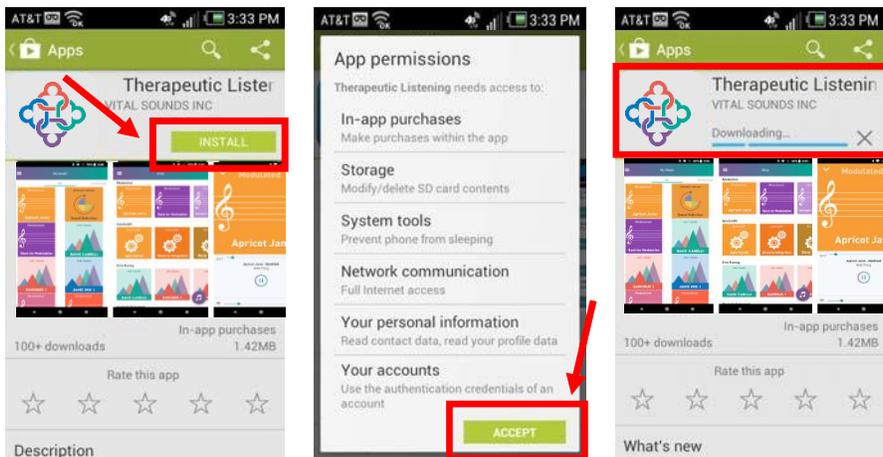


Step 4 第四步

Open the app in one of two ways:
通过下面两种方式打开应用程序

1. Click on "Open" while in the Store
在商店中点击'打开'

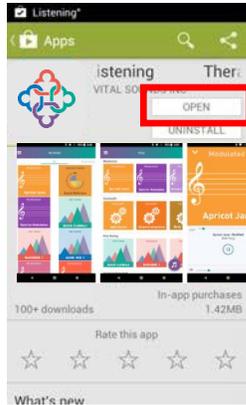

2. Locate the app on your Home Screen and touch the icon to open
在主屏幕上找到应用程序，然后点击图标打开

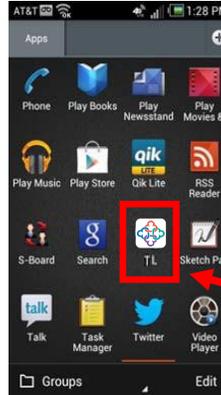


Step 5 第五步

There are two primary areas to navigate:
有两个主要的领域:


"Shop"
商店

&

"My Music"
我的音乐

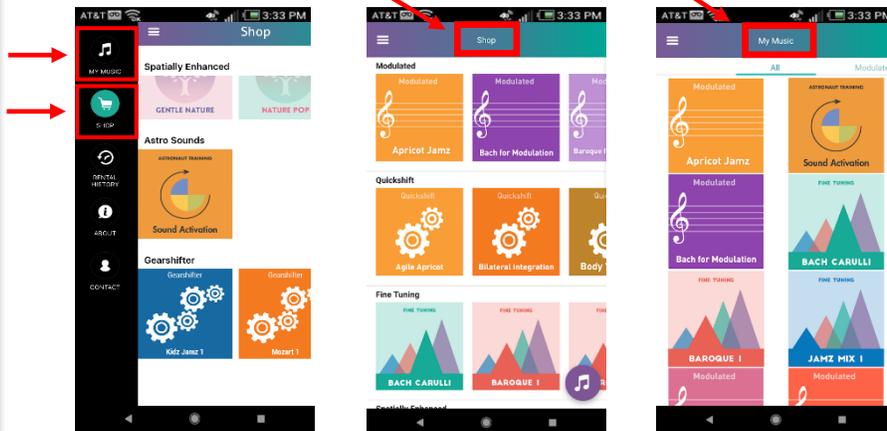

Step 6 第六步

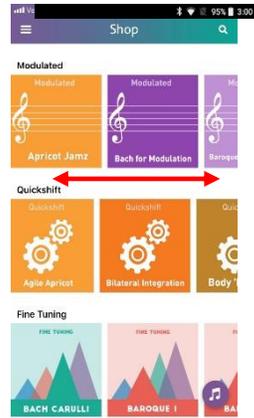


“Shop” is where you can purchase albums. Scroll up/down to locate the Album Category you wish to search. Then, swipe the album images left/right to locate the album you wish to access.
 “商店”是你可以购买专辑的地方。向上/向下滚动找到您希望搜索的专辑类别。然后，向左/向右滑动专辑图片来定位您希望访问的专辑。




Once you locate your album, select the album image to review its details.
 一旦你找到了你的专辑，选择专辑的图片以查看其更详细的信息

Step 7 第七步

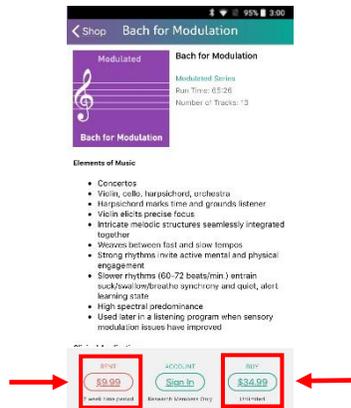


Click the “Buy” button to purchase full access to an album or you have the option to Click “Rent” to purchase access for 2 weeks*
 (*select albums only)
 点击“购买”按钮购买专辑的完整访问权限，
 或者点击“租借”选项购买访问权限 2 周*
 (* 仅限部分专辑)

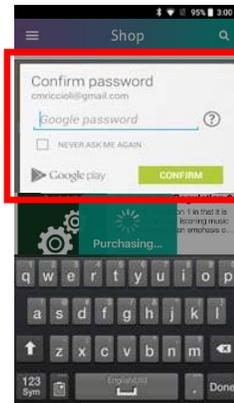


Albums are purchased/rented using your Google Play account. Transactions are considered In-App Purchases.
 使用您的谷歌 Play 账户购买/租借专辑。
 交易将被视作应用内购买。

Step 8 第七步

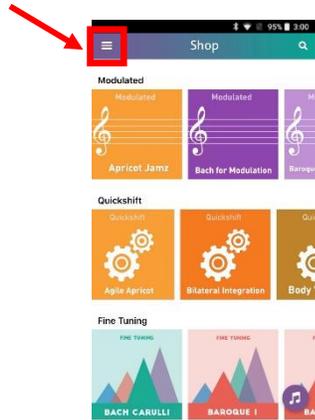

Once purchased, albums will be added to the TL App "My Music" section. To navigate to "My Music," click the menu icon (3 horizontal bars) in the top left. 一旦购买，专辑将被添加到 TL 应用程序“我的音乐”部分。要回到“我的音乐”，点击左上角的菜单图标(3个单杠)。

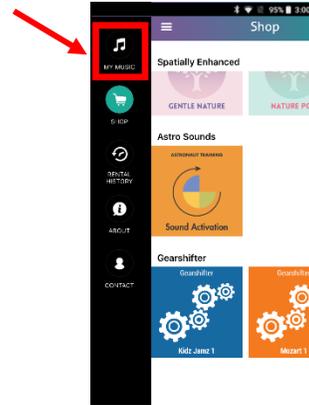

Enter "My Music" to access the album(s). 进入“我的音乐”以使用专辑

Step 9 第九步



In "My Music," you'll find your purchase. Select the album image to enter the play screen.

在“我的音乐”里，你会找到你所购买的专辑。选择专辑图片进入播放界面。



Once downloaded, the "Play" button will appear and you can press it to begin the music selection. 下载完成后，“播放”按钮就会出现，你可以按下它开始聆听音乐选集。

Android FAQs 安卓问答

- Course Completion Number Disclaimer Pop-up Message:
 - To access Modulated Music albums, a Trained Therapeutic Listening Provider Number is necessary.
 - To access Fine Tuning Music albums, an Advanced Therapeutic Listening Provider Number is required.
- 弹出需课程完成号码的免责声明:
 - 要获得调制音乐专辑, 必须有一个受训过的听乐治疗提供者号码。
 - 要获得微调音乐专辑, 需要一个高级听乐治疗服务提供者号码。
- A device must have the ability to download an App from the [Google Play Store](#) in order to access the TL App.
- 为了访问 TL 应用程序, 设备必须有从 [谷歌 Play Store](#) 下载应用程序。
- A device requires Wi-Fi or Data access to download the free TL App and any album purchases.
- 一个设备需要 Wi-Fi 或数据访问才能下载 TL 应用程序和购买专辑。
- A device does not require Wi-Fi or Data access to listen to purchased albums. “Airplane Mode” is recommended to avoid distraction during listening times.
- 一个设备不需要 Wi-Fi 或数据访问来收听购买的专辑。建议使用“飞行模式”, 以避免在听音乐时分心。
- To access an album on multiple devices, the device must be signed into the same Google Play account that the album was originally purchased under.
- 要在多台设备上访问一张专辑, 设备必须登录到最初购买该专辑时所用的同一个谷歌 Play 账户。
- The number of devices a Google Play account can be connected to is determined by Google Play; however, they typically allow up to 10 devices for an account to be signed into simultaneously.
- 一个谷歌 play 账户可以连接的设备数量由谷歌 play 决定; 然而, 他们通常允许一个账户同时登录最多 10 台设备。
- Album purchases are considered In-App purchases, which are not allowed as shareable items within a Google Play Family Library.
- 购买专辑被视作应用内购买, 在谷歌 Play 的家庭图书馆中, 这是不允许共享的。
- Google Play considers rentals as one-time subscriptions, which are not allowed to be shared/synced between devices, regardless of login credentials.
- 谷歌 Play 将租借视为一次性订阅, 不允许在设备之间共享/同步, 无论登录账户是否相同。
- Albums are not transferrable between Android and Apple devices.
- 专辑不能在苹果和安卓设备之间转移。
- Android Tablets and Smart Phones are compatible with Therapeutic Listening headphones.
- 安卓平板电脑和智能手机均与听乐治疗耳机兼容
- Albums are set to shuffle automatically within the app.
- 专辑在应用程序中被自动设置为随机选择。
- Tech Tips 技术贴士:
 - Ensure your device and the TL App are updated to the most current versions.
 - Uninstall and reinstall the TL app to refresh the app.
 - Full purchases will be restored automatically.
 - Further Troubleshooting:
 - Go to <https://vitallinks.com/support/contact/contact-us/> and select “Therapeutic Listening App” as your Submission Topic.
 - 确保你的设备和 TL 应用程序更新到最新版本。
 - 卸载并重新安装 TL 应用程序来刷新应用程序。
 - 完整购买的专辑会自动恢复。
 - 进一步故障排除:
 - 通过 <https://vitallinks.com/support/contact/contact-us/> 并选择“Therapeutic Listening App”作为投稿主题。